# Multisteril Fast

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EN

Tecno-Gaz S.p.A. has been able to create **a new and revolutionary device that can automatically carry out all the major preparation phases for the sterilization cycle, such as disinfection, cleansing, rinsing, drying, without human intervention, without any professional risk and with a reduction in the overall time taken.** A device that is unique in the world and indispensable for every facility that carries out sterilization.

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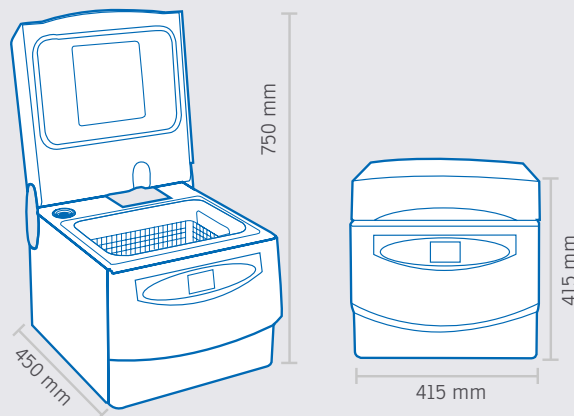
**The aspects that Multisteril Fast can help you manage correctly**

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### Assistants' expenditure of time

Assistants spend a lot of time for the preparation phase to tools sterilization, because of multiple and complex manual procedures. **Multisteril Fast does not require human presence.**

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### Risk factor

The manipulation of tools and potential infectious materials is a very important factor and it must be kept under strict control, both for operators' protection and the responsibility of the owner of the studio. **Multisteril Fast cancels the risk.**

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### Management protocol

There is the sterilization if the procedure of preparation of materials is done properly, if not, the whole process is completely nullified. **Multisteril Fast performs correctly all the cycles.**

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### Tested disinfection

Tested issued by University of Dresden attesting cleaning **capacity of Multisteril Fast in combination with Multisteril CD cleaner.**

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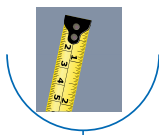
### Spaces

Space is another important problem, in fact, sterilization rooms often have small size and rationalization of space assumes a considerable value. **Multisteril Fast can do all the process filling 40 cm.**

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### Overall costs

Cost is an important component which must be brought under control. **With Multisteril Fast every cycle costs about 2 Euros.**

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Liquid  
concentrate for  
the non  
protein-fixing  
cleaning and  
disinfection

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Liquid concentrate for the non  
protein-fixing cleaning and disinfection

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Support for 2 surgical trays  
(article code refers  
to support only)

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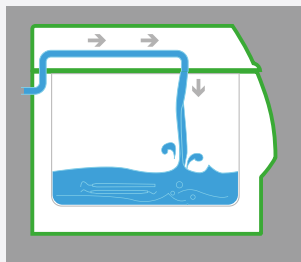
**Stand-by**

To improve cleansing, an initial standby phase has been created, which allows the tub to be filled with water and liquid detergent/disinfectant, so that the instruments sit better in the solution, **ensuring perfect cleansing, even when using bi-component cements.**

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
**Loading the water**

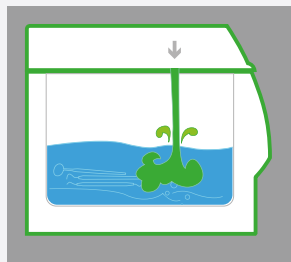The water enters the tank thanks to the **direct connection to the mains water supply.**

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
**Disinfectant/detergent entry and liquid activation**

A special peristaltic pump present inside the device automatically **introduces the percentage of disinfectant/detergent previously outlined in the parameters of the cycle.**

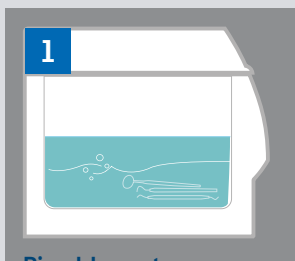Fasi del processo | Phases of the process

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**Heating**

The device moves on to heating and to a **first static decontamination of the instruments.**

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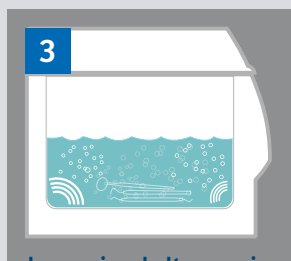
**Degasification**

This process **eliminates the gases dissolved in the water** avoiding interference with the ultrasounds and improving efficiency.

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
**Ultrasonic wash**

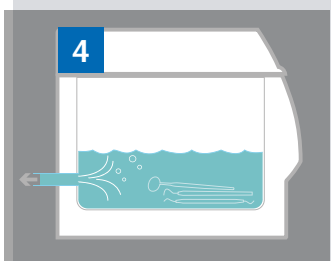At the end of the degasification phase the ultrasonic cleansing begins, setting off the cavitation, which allows for the removal of chemical and organic residue present on the materials immersed in the tank, **destroying the bacterial load by more than 95%. The length of this procedure can be programmed by the operator for between 5 and 30 minutes.**

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
**Waste with pump**

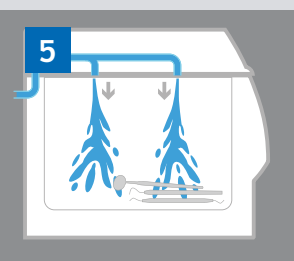At the end of the cleansing phase, **MultiSteril automatically empties all the liquids present in the tank.** Once the tank is completely empty the following rinse phase begins.

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
**Rinse**

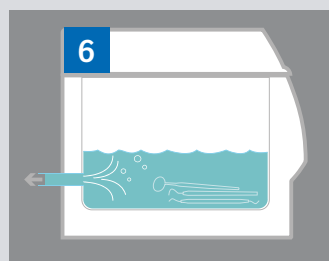To remove all chemical residue from the instruments, **the rinse phase begins, taking place through the pressurised intake of water through the special nozzles in the lid.** The water taken in is then automatically removed through the mesh with a special pump that permit to make the process faster. At the end the drying phase is launched.

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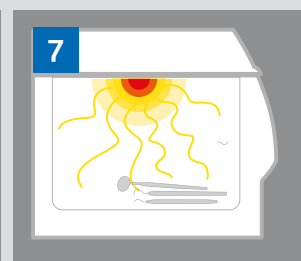
**Drying**

To guarantee a proper drying of the materials **the machine generates a flow of hot air, produced by a fan and a resistor located on the lid of the device,** which lightly touches the objects and takes away the dampness in the form of gas.